
Microstream[®]


ETCO₂ SAMPLING

**Products for Intubated
Patients
Adult and Pediatric Patients
Directions for Use**


FilterLine[®] Sets

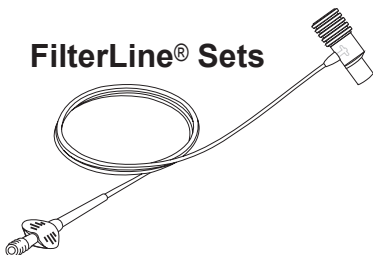FilterLine[®] H Sets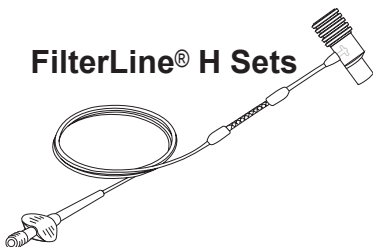

VitaLine[™] H Sets

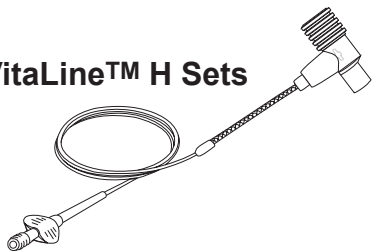This product is covered by one or more of the following US patents: 6,422,240; 5,857,461; 6,437,316; 6,926,005; 5,657,750; 7,383,839; 7,500,483 and their foreign equivalents. Additional patent applications pending.

© 2011 Oridion Medical 1987 Ltd.
All rights reserved.

PN MQ04616I



Microstream® EtCO₂ Sampling for Intubated Patients

Instructions for use:

1. Firmly connect the small-end (male) of the Microstream® Airway Adapter **1** to the female-end of the Wye piece (ventilation source) .
2. Firmly connect the patient's endotracheal tube connector into the large-end (female) **2** of the Microstream® Airway Adapter.
3. Ensure that the tubing is not twisted or crimped.
4. Twist the female luer connector **3** securely but gently into the Microstream® input connector **4** .
5. Following connection of the CO₂ sampling line, check that CO₂ values appear on the monitor display.

Instructions for use with a closed suction system:

1. Follow step 1 as described above
2. Firmly connect the large-end (female) **2** of the Microstream® airway adapter to the ventilator connector on the closed suction system, so that the airway adapter does not interfere with the functioning of the suction catheter.

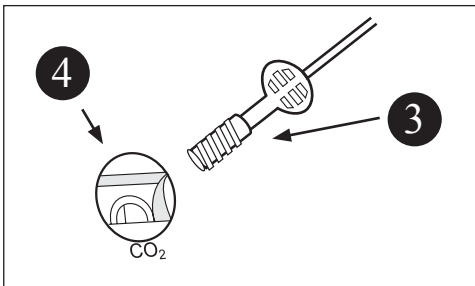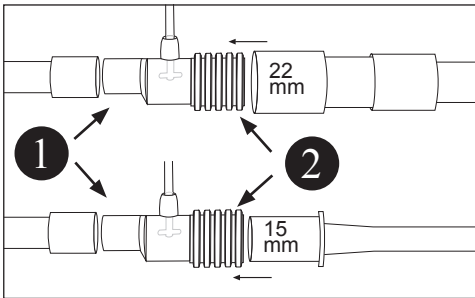ENG

Microstream® EtCO₂ Sampling for Intubated Patients Adult and Pediatric Patients

FilterLine® Sets for short-term use

FilterLine® H Sets for humidified ventilation
and long-term use

VitaLine™ H Sets for ambient humid environments up to 75% ambient relative humidity

Airway Adapters for Adult to Pediatric patients:

- Added dead space <6.6 cc.
- To be used with an endotracheal tube > 4.5mm bore.
- Suitable for use with 22mm and 15mm patient tubing.

For use with all Microstream-enabled monitors.

Single patient use, not to be re-processed.

Non-sterile / non-pyrogenic.

Latex free

Warning: When used with closed suction system, do not place the airway adapter between the suction catheter and endotracheal tube. This is to ensure that the airway adapter does not interfere with the functioning of the suction catheter.

Warning: Loose or damaged connections may compromise ventilation or cause an inaccurate measurement of respiratory gases. Securely connect all components and check connections for leaks according to standard clinical procedures.

Caution: Do not attempt to clean, disinfect, sterilize or flush any part of the sampling line as this can cause damage to the monitor.

Caution: Dispose of sampling lines according to standard operating procedures or local regulations for the disposal of contaminated medical waste.

Note: During nebulization or suction, in order to avoid moisture buildup and sampling line occlusion, remove the sampling line luer connector from the monitor.

Note: Replace the sampling line according to hospital protocol or when a blockage is indicated by the device. Excessive patient secretions or a build-up of liquids in the airway tubing may occlude the sampling line, requiring more frequent replacement.

Note: When connecting a sampling line to the monitor, insert the sampling line connector clockwise into the monitor CO₂ port and turn the connector until it can no longer be turned, to ensure that it is connected securely to the monitor. This will assure that there is no leak of gases during measurement at the connection point and that measurement accuracy is not compromised.

Note: Following connection of the CO₂ sampling line, check that CO₂ values appear on the monitor display.

Note: During setup, ensure that airway adapter can be easily attached and detached from the tubing before proceeding.

Federal law (U.S.) restricts this device to sale by or on the order of a physician.

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Obelis S.A.
Oridion Capnography Inc.