

eFONA\* in CICO situation

According to DAS 2015 guidelines

Compact and lightweight set



## ScalpelCric


Set for surgical cricothyrotomy according to the scalpel technique

# / ScalpelCric

ScalpelCric is a surgical set for cricothyrotomy according to the scalpel technique. The set has been developed to match the latest recommendations from the DAS (Difficult Airway Society) 2015 Guidelines.

Based on the simple description of "**stab, twist, bougie, tube**", the ScalpelCric is meant to facilitate the execution of eFONA (emergency Front Of Neck Access) in a CICO situation (Cannot Intubate – Cannot Oxygenate). The set is packed in a specific sterile pouch with each component organised in the order of use. This can be important at the critical time of the procedure.

## Use

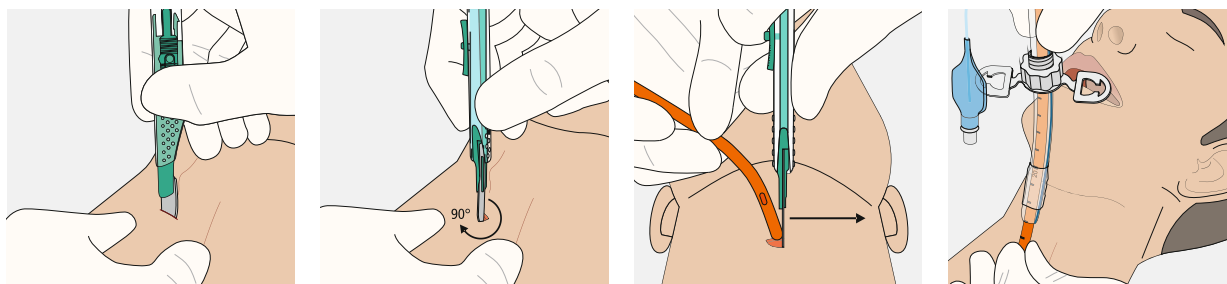

1. Stab

2. Twist

3. Bougie

4. Tube

## Features

### Bougie

- Low friction surface for easy railroading of the tube
- Angled and atraumatic tip
- Centimetre markings to control insertion depth

### 6.0 mm I.D. tube

- Cuff for protection against aspiration
- Moveable flange
- Atraumatic, rounded tube tip

### Extension tubing

- With suction possibility  $\leq 16$  Fr

### Necktape

- Safe fixation of the tube



1 2 3 4 5 6

### Content

- 1 Scalpel # 10
- 2 14 Fr Bougie (40 cm)
- 3 6.0 mm I.D. tube with cuff
- 4 Extension tubing
- 5 Syringe 10 ml
- 6 Necktape

Robust packaging and compact pack size to fit emergency bags, backpacks, crash carts, etc.

### Pack size:

31.5 cm (L) x 15.2 cm (W) x 5.0 cm (H)

### Unfolded size / sterile field:

56 cm (L) x 39 cm (W)

## Order information

ScalpelCric / For single use, sterile

REF Box

30-08-447-1 1

## Additional information



Application Video



Flyer  
Cricothyrotomy



Annotated  
Bibliography

\* emergency Front Of Neck Access

This product is manufactured without the use of natural rubber latex, unless otherwise specified.  
This product does not contain phthalates which require labelling according to CLP Regulation (EC) 1272/2008.