

## Ultimate Puncture Control




### New Medlance<sup>®</sup> Plus Safety Lancet

*Safe and Simple*

#### INSTRUCTIONS OF USE:

Medlance<sup>®</sup> plus is very easy to operate, which makes blood micro-sampling comfortable for patients and professionals. Follow the simple steps listed below and always observe safety procedures of your local health authority for disposal.

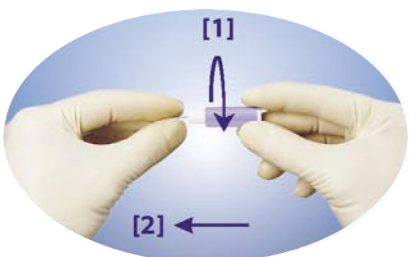

Twist off the purple (or blue) protective cap, and then pull it straight out.



Press the Medlance<sup>®</sup> plus body firmly against the puncture site to activate the device.



Gently apply intermittent pressure near the puncture site to obtain the required blood volume.

PRODUCT	NEEDLE/BLADE WIDTH	PENETRATION DEPTH	RECOMMENDATION FOR USERS	BLOOD FLOW
<b>Medlance<sup>®</sup> plus Lite</b>	<b>Needle 25 G</b>	<b>1.5 mm</b>	Collecting blood has never been so painless! Medlance <sup>®</sup> plus Lite is ideal for blood glucose monitoring purposes	
<b>Medlance<sup>®</sup> plus Universal</b>	<b>Needle 21 G</b>	<b>1.8 mm</b>	Whenever you need larger blood samples for testing glucose, hemoglobin, cholesterol, blood type studies, coagulation, blood gas and many others	

FOR MORE INFORMATION AND ORDERING PLEASE CONTACT:

Cambridge Sensors USA, LLC  
 toll free: 877.374.4062 • [www.microdotcs.com](http://www.microdotcs.com)  
 10051 Bode Rd. Plainfield, IL 60585