

VividTrac[®]

Single Use Video Intubation Device (adult & pediatric)



Open System - Single Use Video Intubation Device, utilizing standard USB II interface that works on a wide variety of computers and tablets, with NO proprietary system cost.

Superior Imaging - Wide Angle, High Resolution, Digital Video of airway displayed on the computer screen allows the user to *store, display on TV, transmit or broadcast* over WiFi or 3G networks, with off-the-shelf computing and connectivity solutions.

NO Head Tilt - Blade tip positions easily at the epiglottic vallecula to facilitate direct visualization of the cords, with the patient in any position.

NO Stylet and Concurrent Suction - Integrated ETT Channel guides the ETT precisely through the vocal cords and trachea without the need for a Stylet, allowing for *concurrent suction* using the free right hand.


NO Batteries - 100% powered through USB cable.

Anti-fog - Automatic anti-fogging camera.

Portable and Affordable - NO cleaning, NO maintenance, we recycle.

Vivid Medical Inc.

Vision to Help Save Lives

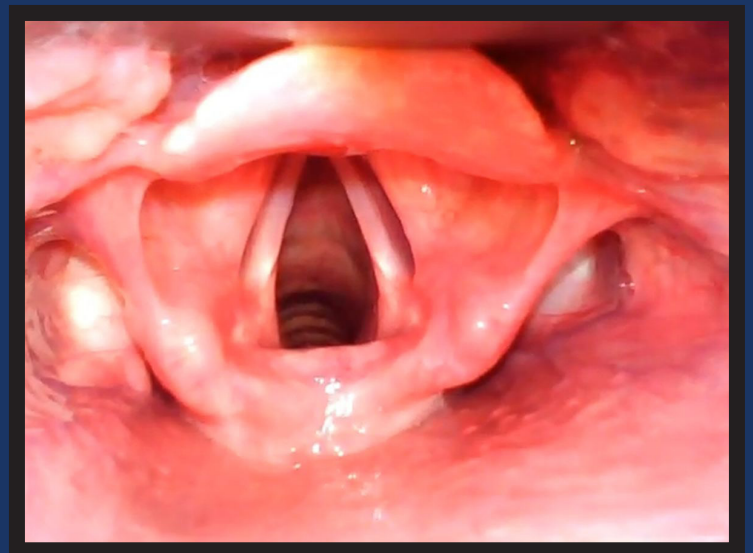



Vivid Medical, Inc.

www.VividMed.com

sales@vividmed.com

+1 650 618 5715

VividTrac Adult	VividTrac Pediatric
	
VividTrac VT-A100 (adult) VividTrac VT-A105 (adult)	VividTrac VT-P100 (pediatric) VividTrac VT-P105 (pediatric)
ET Tube Sizes	ET Tube Sizes
6.0mm to 8.5mm	4.0mm to 6.0mm

VividVision® Tablets

Overview









VividVision is a software application designed specifically for the Windows 8 and Android touch screen tablets in support of Vivid Medical Vision Devices such as the VividTrac (single-use video intubation device). VividVision Tablets provides feature rich options for display and recording of Vivid Medical's Vision Devices, in the form of a low cost and portable solution. The application when running has the capability to auto-detect our Devices (VividTrac) upon plug-in, and to immediately display the high resolution video on the wide viewing angle, and bright LCD display of the tablet. VividVision tablets provide a Complete, Immediate and On-the-Go solution for the use of Vivid Medical's Vision Devices.



VividVision Windows 8 on 10" Tablet

Software Features

The following are the features supported by the VividVision application.

Icon	Feature	Description
	Snapshot Icon	Touching the Snapshot icon takes an image snapshot of the displayed. Media is saved in standard JPG image format on the tablet's internal storage.
	Record Icon	Touching the Record icon starts and stops the video recording. Media is saved in standard MP4 video format on tablet's internal storage. Icon displays RED while recording.
	Gallery Icon	Touching the Gallery icon displays a gallery view of the captured images and recorded video files.
	USB Icon	USB icon turns automatically GREEN when a supported USB Vision Device is connected to the USB port of the tablet using an OTG (On-The-Go) cable. USB Icon displays RED when no USB Vision Device is detected.
	Battery Level Icon	Battery Level icon displays the percentage of battery charge remaining on the tablet. The % text color changes to YELLOW and RED as the battery charge diminishes. (not available on Windows 8)
	Storage Level Icon	Storage Level icon displays the percentage of storage space available to support additional media storage (image or video capture). The % text color changes to YELLOW and RED as the free storage space reduces on the tablet. (not available on Windows 8)
	Setting Icon	Touching the Settings icon displays a pop up window indicating the application parameters that can be changed and set by the user (see "Settings" section below for details).
	Exit Icon	Touching the Exit icon will exit the VividVision application.

Software User Settings

The following are the User settable parameters of the VividVision application.

- **Display in fullscreen mode** – When enabled the application will startup in full screen video image viewing mode and all control icons will be hidden. Full screen mode can be enabled or disabled at any time by touching the tablet screen anywhere on the video image.
- **Start Recording Video When Device is Connected** – When enabled video recording will start automatically as soon as a supported device is connected, and will continue to record until the Vision device is disconnected or the User touches the Record Icon to manually stop the recording.
- **Filename <TAG>** - The filename "TAG" will be included at the beginning of the filename for all captured images and recorded video files. Note, all filenames of captured media will also contain the <Date> and <Time> the media was captured.

Medical Rolling Tablet Stand

For the hospital, the VividVision Tablet can be mounted on a rolling medical stand, thus allowing mobility of a display device mounted and available throughout a department. The attached basket allows for storage of quickly available pouched adult and pediatric VividTrac devices.

Features:

- Easy grip handle
- Wire basket with dual side cord wraps
- Easy rolling wheels, with locks for additional stability



Vivid Medical, Inc.

1023 Corporation Way
Palo Alto, CA 94303
650.618.5715
www.VividMed.com