# Genius™ 2 Tympanic Thermometer

## Instruction For Use

### Taking A Temperature

**STOP** Always apply probe cover prior to insertion in ear canal.

Inspect the patient's ear anatomy for proper probe placement. (No blood or fluid should be present.)

Remove the thermometer from the base.

Inspect the probe lens. It should be shiny and free of cracks or debris without a probe cover attached. (Clean lens with a dry lint-free swab or lens wipe if needed.)

Attach probe cover by aligning the silver tip with a probe cover and push straight down until you hear a snap.

Inspect the membrane on the end of the probe cover. If a hole or tear is visible, discard and attach new probe cover.

Ensure the probe cover is flush with the blue end cap. The silver probe should not be showing. A probe cover **MUST** be used.

Insert the tapered probe tip. Align the probe at the entrance of the patient's ear canal.

Immediately press the large triangular scan button located in the center of the handle. A triple beep will sound when completed.

Dispose of the probe cover into a waste receptacle by pressing the eject button.

Return the thermometer to the base unit for storage.

LCD display will go into "sleep" mode after 10 seconds of inactivity.

To recall the last temperature, press and release the scan button before loading another probe cover. Loading a new probe cover will erase the last temperature taken from memory.

### Optional Functions

To change the temperature display press and hold the C/F° button for 2 seconds when the LCD screen is active.

To use the 60 second pulse timer, press and hold the (clock icon) for two seconds to activate.

Press and release the (clock icon) once to begin (beeps will be heard at 15 second intervals) and it will automatically shut off at 60. If finished with function prior to complete 60 seconds, press and release (clock icon) a second time to deactivate function.

Clean **ONLY** with approved cleaning agents, **WIPE** to clean.

**COVIDIEN**

positive results for life™

COVIDIEN, COVIDIEN with logo and ™ marked brands are trademarks of Covidien AG or its affiliates.

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©2008 Covidien AG. All rights reserved.