

OPTI-CIDE³[®]

Disinfectant Cleaner/Sanitizer



Use this product to clean and disinfect hard, non-porous, inanimate surfaces such as those made of acrylic, metals (aluminum, brass, stainless steel, chrome), glass, plastics (polycarbonate, polypropylene, polystyrene and polyvinylchloride), clear plastics, Plexiglas[®], vinyl as well as laminated and painted surfaces. Opti-Cide³[®] effectively and quickly loosens and suspends blood and bodily fluids, non-organic and organic soil, and other cellular debris for easy removal.

- **2-Minute hard, non-porous surface disinfectant**
- **10-Second soft surface sanitizer (Solution only)**
- **Tuberculocidal, Virucidal, Fungicidal, Bactericidal**
- **Ready-to-Use**

Order No.	Size	Order No.	Size
OCS12-024	24 fl. oz. (710 mL) Spray	OCP01-1920	15 Gallon (57 L)
OCP12-024	24 fl. oz. (710 mL) Pour	OCP01-3840	30 Gallon (114 L)
OCP04-128	1 Gallon (3.8 L)	OCP01-7040	55 Gallon (208 L)
OCP02-320	2.5 Gallon (9.5 L)	OCW06-100	100 Wipes

Disinfectant Cleaner/Sanitizer

Medical/Surgical/Hospital Use: 2-minute contact time

Acinetobacter baumannii
Avian Influenza A Reassortant Virus (H3N2)
Escherichia coli – with ESBL (*E. coli*)
Hepatitis B Virus (HBV)
Hepatitis C Virus (HCV)
Herpes Simplex Virus (HSV-2)
Human Immunodeficiency Virus (HIV-1)
Swine Influenza A Virus (H1N1)
Influenza A Virus Strain A2/Hong Kong
Carbapenem-resistant *Klebsiella pneumoniae* (CRKP)
Listeria monocytogenes
Mycobacterium bovis BCG (tuberculosis) (TB)
Pseudomonas aeruginosa
Rotavirus Strain WA
Salmonella enterica
Serratia marcescens
Staphylococcus aureus
Methicillin-Resistant *Staphylococcus aureus* (MRSA)
Trichophyton interdigitale (ringworm)
Vancomycin-Resistant *Enterococcus faecalis* (VRE)

Veterinary/Pet Care Facility Use: 3-minute contact time

Bordetella bronchiseptica (kennel cough)
Bovine Viral Diarrhea Hepatitis C Virus
Duck Hepatitis B Virus
Canine Distemper Virus
Canine Para-Influenza Virus
Chlamydia psittaci (parrot fever)
Feline Leukemia Virus
Infectious Bovine Rhinotracheitis Virus
Murine Norovirus
Streptococcus equi
Canine Parvovirus (5 minute contact time – wipes only)

Procedures

Hospital and General Disinfection Solution Directions for Use

Directions:

It is a violation of Federal law to use this product in a manner inconsistent with its labeling. Apply this undiluted product by sponge, rag, cloth, mop, spray or immersion to wet all surfaces thoroughly.

For surface cleaning:

To clean surfaces, saturate surfaces with this product by spray, mop, rag, towel, paper towel, pour, or immersion then wipe surface clean. Repeat, if necessary, until surfaces are visibly clean. Discard paper towels or launder cloth towels before reusing.

For use as a hard, non-porous non-food contact surface disinfectant/virucide/fungicide:

Apply this product to pre-cleaned surfaces by sponge, cloth, microfiber, mop, spray, pour or immersion to wet surfaces thoroughly. Allow to remain wet for 2 minutes at room temperature, then wipe dry using a microfiber mop, clean paper or cloth towel, or rinse surface using potable water and wipe surface dry or allow to air dry. Discard or launder spent materials before reuse.

Procedures

Veterinary/Pet Care Solution Directions for Use

Directions:

It is a violation of Federal law to use this product in a manner inconsistent with its labeling. Apply this undiluted product by sponge, rag, cloth, mop, spray or immersion to wet all surfaces thoroughly.

For use as a hard, non-porous non-food contact surface disinfectant/virucide/fungicide:

Apply this product to pre-cleaned surfaces by sponge, cloth, microfiber, mop, spray, pour or immersion to wet surfaces thoroughly. Allow to remain wet for 3 minutes at room temperature, then wipe dry using a microfiber mop, clean paper or cloth towel, or rinse surface using potable water and wipe surface dry or allow to air dry. Discard or launder spent materials before reuse.

Procedures

Hospital and General Disinfection Wipes Directions for Use

Directions:

It is a violation of Federal law to use this product in a manner inconsistent with its labeling. **Do not use this product as a baby or skin wipe, or on contact lenses!** Pre-clean all surfaces to be treated with this product. Apply this undiluted product by sponge, rag, cloth, microfiber, mop, spray or immersion to wet surfaces thoroughly.

For surface cleaning:

To clean surfaces, use a fresh wipe on surfaces. Repeat as necessary until surfaces are visibly clean.

For use as a hard, non-porous non-food contact surface disinfectant/virucide/fungicide:

Pre-clean hard, non-porous surfaces to be disinfected with a clean wipe. Use a fresh, clean wipe to disinfect hard, non-porous surfaces until thoroughly wet. Allow surfaces to remain wet for 2 minutes at room temperature.

Procedures

Veterinary/Pet Care Wipes Directions for Use

Directions:

It is a violation of Federal law to use this product in a manner inconsistent with its labeling. **Do not use this product as a baby or skin wipe, or on contact lenses!** Pre-clean all surfaces to be treated with this product. Apply this undiluted product by sponge, rag, cloth, microfiber, mop, spray or immersion to wet surfaces thoroughly.

For use as a hard, non-porous non-food contact surface disinfectant/virucide/fungicide:

Pre-clean hard, non-porous surfaces to be disinfected with a clean wipe. Use a fresh, clean wipe to disinfect hard, non-porous surfaces until thoroughly wet. Allow surfaces to remain wet for 5 minutes at room temperature.

This product cannot be used to disinfect critical or semi-critical medical devices.